

Puerperal Uterus Palpation Training Model

LM-055

Instruction Manual

Thank you very much for purchasing the Puerperal Uterus Palpation Training Model.
Please read this instruction manual carefully to ensure correct use of the product,
and store it in a safe place for easy access.

Manufacturer:

KOKEN CO., LTD.

1-4-14 Koraku, Bunkyo-ku, Tokyo 112-0004 JAPAN

www.kokenmpc.co.jp





Handling and Safety Precautions

These precautions should be strictly observed in order to ensure safe, long-term use of the product.

The following precautions should be observed particularly strictly:


1. Since this model is made to give the same sensation to a living body, unreasonable handling may lead to breakage of the model.
2. Pinching or beating the abdomen may lead to breakage of the model.
3. Use a diluted neutral detergent to remove dirt on the model; if this is not sufficient, use a gauze soaked with ethanol for disinfection to wipe off.
※ Use of solvents such as a thinner or benzene is prohibited.
4. Do not place anything on the abdomen; the model is deformed.
5. If the epidermis or uterus recession models are felt oily, apply the attached baby powder.

1. The contents of this instruction manual are subject to change without notice.
2. This instruction manual may not be reproduced in part or in its entirety without permission.
3. Please contact the manufacturer in the event that any errors or omissions are found in the contents of this instruction manual.
4. This product should be used only as described in this instruction manual. In particular, the product should not be used in any way that contravenes the precautions noted in the instruction manual.

Table of Contents


1. Name of each part
2. Instructions for handling
3. Storage method
4. Specifications
5. Accessories

1. Name of each part

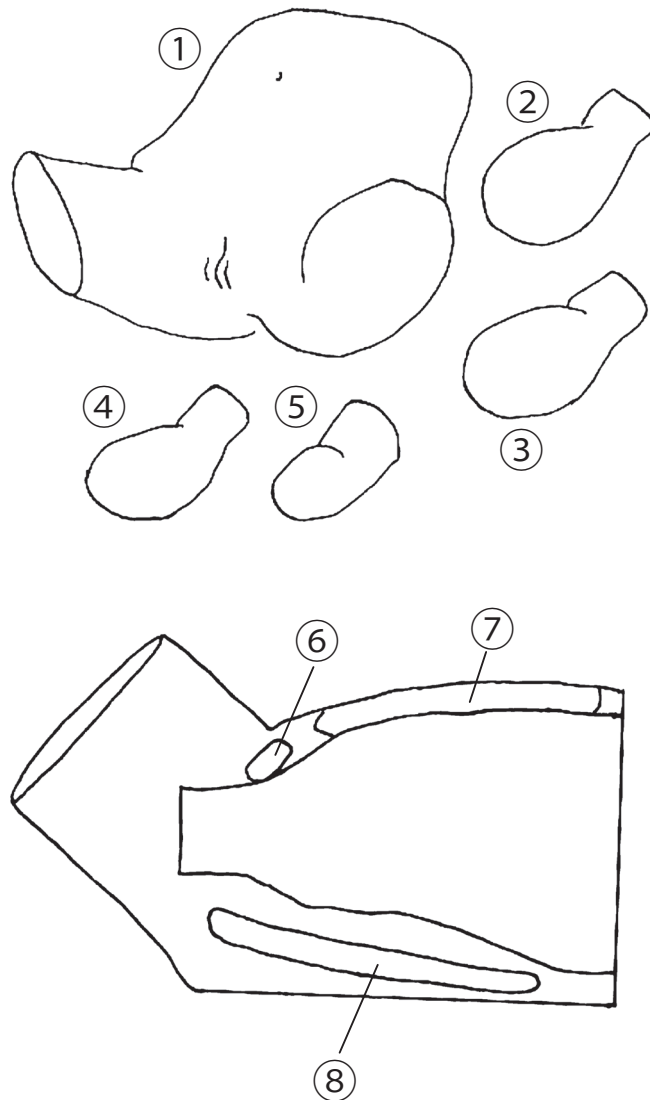

- ① Main body
- ② Uterine Involution (Normal, Day 1)
- ③ Uterine Involution (Subinvolution)
- ④ Uterine Involution (Normal, Day 3)
- ⑤ Uterine Involution (Normal, Day 5)
- ⑥ Pubis
- ⑦ Abdominal wall
- ⑧ Spine

2. Instructions for handling

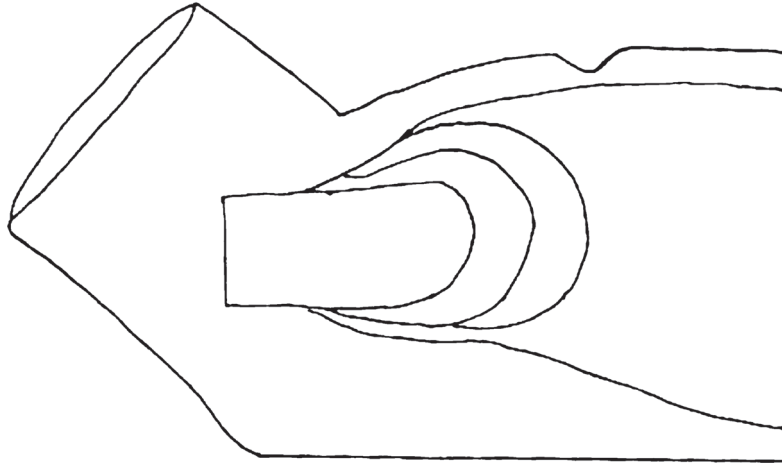

Method for setting the Uterine Involution Models

1. Apply enough powder to the abdominal cavity of the Uterine Involution Models.
2. Set the model in the position shown in the figure.
 - ※ Make sure that the model is not dislocated as the direction is fixed.

3. Storage method

1. Store the main body with the attached cover in a place avoiding direct rays of the sun or ultraviolet rays.
2. Place the attached sponge for prevention of deformation in the abdominal cavity of the main body.
3. Store the Uterine Involution Models in the attached bag.
4. When the attached bag get wet (with rain), the contents may be stained with dyes oozing out from the lining.

4. Specifications

Item	Length	Width	Height	Weight
Main body	Approx. 44 cm	Approx. 48 cm	Approx. 27 cm	Approx. 4.4 kg
Uterine Involution (Normal, Day 1)	Approx. 21 cm	Approx. 14 cm	Approx. 11 cm	Approx. 1 kg
Uterine Involution (Subinvolution)	Approx. 21 cm	Approx. 14 cm	Approx. 11 cm	Approx. 470g
Uterine Involution (Normal, Day 3)	Approx. 16 cm	Approx. 11 cm	Approx. 9 cm	Approx. 660g
Uterine Involution (Normal, Day 5)	Approx. 14 cm	Approx. 7 cm	Approx. 7 cm	Approx. 470g

5. Accessories

Product	Quantity	Reference
Cloth	1	For the main body
Sponge for prevention of deformation	1	For the main body
Storage bag for uterus	1	For the Uterine Involution Models
Powder	1	