

4-Month-Old Baby, Sho-Chan

LM-075

Instruction Manual

Thank you for purchasing the 4-Month-Old Baby, Sho-Chan.
Please read this instruction manual carefully to ensure correct use of the product, and store it in a safe place for easy access.

KOKEN CO., LTD.

1-4-14 Koraku, Bunkyo-ku, Tokyo 112-0004 JAPAN

URL:<http://www.kokenmpc.co.jp>

E-mail:customer@kokenmpc.co.jp

C-1-991-1-02-00



Handling and Safety Precautions

These precautions should be strictly observed in order to ensure safe, long-term use of the product

Precautions when using the product

- When rotating an arm or leg of the doll, grip it at the base of the arm or leg.
- Do not rotate or pull the neck or an arm or leg of the doll with excessive force. Doing so might damage the doll.
- The doll is moderately heavy. Do not hold it just by the arm or leg since this might cause the arm or leg to come off.
- If an arm or leg comes off, warm the joint of the arm or leg a little using a dryer. Doing this allows you to easily re-install the arm or leg on the doll.
- Using a nail or sharp-edged tool might damage the doll.
- Do not put the doll in the bath or allow the doll to become wet or moist.
- Do not leave the doll inside the car in the hot summer sun or in locations with high temperatures.
- Do not drop or disassemble the doll. Doing so might damage the doll.
- Never use oil-based inks, paints, or other substances on the model.
- Never put the doll on printed paper or flexible plastic.
- To clean the surface of the doll, apply neutral detergent (dishwashing detergent, soap, etc.) and wipe away lightly with damp cloth. Do not use alcohol or other organic solvent. Rubbing the surface of the doll with excessive force might cause paint on hair of the head, lip, or cheek to peel off.
- Clean the doll with wet towel etc. Always keep the doll clean.
- For safety reasons, do not use the doll once it has become damaged or deformed.

Storage Method

- When storing the doll, place it in the upright position. Do not place the doll upside down or place a heavy object on the doll, as doing so might cause the doll to deform.
- Avoid storing the doll at high or low temperatures.